



# Proven technology, reliable performance

## Getinge 46-series Washer-Disinfector

This document is intended to provide information to an international audience outside of the US.

GETINGE 



# A strong reputation for performance

## Choose the washer- disinfector that's trusted worldwide

When it comes to patient care, downtime can be dangerous. A reliable washer-disinfector keeps your facility running smoothly, protecting patients from cross-contamination.

Getinge 46-series Washer-Disinfectors have been proven in hospitals worldwide. They offer high uptime and customer satisfaction. These washer-disinfectors process most kinds of reusables – from complicated instruments to ward utensils, with an intensive wash program for heavily soiled instruments. Even most kinds of tubular instruments can be cleaned, disinfected, and dried.

The Getinge 46-series includes models for wards, outpatient clinics, surgical departments, and central sterile supply departments (CSSD). Choose between single-door or double-door models to adapt to the capacity, depending on the type and quantity of items you need to process.

# Adapts to your needs

## A model for every department

Getinge 46-series Washer-Disinfectors are a reliable solution to meet your facility's needs.

### High washing capacity

Getinge 46-series Washer-Disinfectors offer a large chamber size with a minimal external footprint.

The Getinge 46-5 model has space for up to five levels of instrument wash carts. Getinge 46-4 has space for up to four levels of wash carts.

### Effective and economical drying

Thanks to the built-in heat exchanger, the 46-series dries items effectively on both the inside and outside. Drying time and temperature can be preset for each program via the control panel.

### Easy to use

The display uses plain language to explain what's happening in every stage of the washing and disinfecting processes. Simply select one of the six preset programs and press the start button.



# Design with function

The design of the Getinge 46-series is based on function and reliability in mind.

## Attention to design details

The Getinge 46-series Washer-Disinfector includes a number of features that improve usability for staff:

- **Efficient use of space** – A folding door frees floor space while improving ergonomic positioning of the wash cart
- **Ergonomic handle** – An easy grip makes it simple to open and close the door
- **Easy-to-clean surfaces** – Minimize the risk of trapped dirt and pathogens with an easy-to-wipe smooth surface
- **Easy to service** – The 46-series is easily accessible from the front for service access

## Flexible loading equipment

The loading equipment for Getinge 46-series takes flexibility to the next level. Easily customizable wash carts ensure individual processing needs are met perfectly and adapt the same wash cart for many different purposes. Just rearrange it for the sizes, volumes, or types of load you need to process for the moment. It is done in no time – and no tools are needed. Accessories are available for reprocessing surgical instruments, MIS instruments, anesthesia equipment, OP shoes, laboratory glassware, and much more.

The flexibility of the system means fewer carts, which saves both money as well as space in the facility. As your needs change over time, nothing is easier than to complement your wash carts with additional MIS modules. For further flexibility, a loading trolley can be used when changing the type of wash cart.



A two-level wash cart optimized for MIS modules gives flexibility and better access while loading instruments. Both levels can be used for DIN trays or MIS modules.



# Maximum cleaning

## Traceable performance

### Documented proof of cleanliness

The best practices of your CSSD are on the front line in the never-ending battle against hospital-acquired infections (HAIs). The ability to efficiently document and recall sterility data provides assurance for regulatory agencies, accreditation organizations, and an increasingly well-informed public.

Real-time data is a valuable source of information for CSSD managers and hospital administrators to help quickly remediate any problems, improve production planning, and verify compliance. You can securely access cleaning data on demand with the encrypted Getinge Online portal. Getinge 46-series Washer-Disinfectors also integrate with T-DOC, our sterile supply management solution.



### Trusted performance, independent verification of cleanliness

Our Getinge Clean Detergents have provided maximum performance along with efficient and economical throughput for nearly 80 years.

The comprehensive range of instrument cleaning detergents and maintenance products include state-of-the-art formulations to handle the most challenging of instrument cleaning obstacles: pretreatment, complex minimally invasive instrumentation, prion (Creutzfeldt-Jakob disease), and biofilm.

With our central dosing system, the Getinge Clean Management System (CMS), you can be certain that you're using the correct level of detergent in your washer-disinfectors, ultrasonic cleaners, and manual cleaning stations.

Routine use of Getinge Assured Wash Monitoring products provides you with independent verification that the cleaning process is effective and consistent. We offer a full range of residual protein tests for surfaces and instruments – including hollow lumens and scopes.



# Sized to meet your challenge

The Getinge 46-series Washer-Disinfector is available in two models with a capacity of up to 10 DIN trays.

	Units	46-4	46-5
<b>External dimensions</b>			
Width	mm / in	650 / 25.6	650 / 25.6
Depth	mm / in	690 / 27.2	690 / 27.2
Height (with and without dryer)	mm / in	1,860 / 73.2	1,930 / 76
<b>Internal dimensions</b>			
Wash levels (up to)	pcs	4	5
Chamber volume (gross)	L / Cu ft	280 / 9.88	305 / 10.77
Chamber volume (effective)	L / Cu ft	201 / 7.1	225 / 7.94
Chamber width (effective)	mm / in	550 / 21.65	550 / 21.65
Chamber depth (effective)	mm / in	620 / 24.4	620 / 24.4
Chamber height (effective)	mm / in	590 / 23.22	660 / 25.98
DIN 1/1* trays/load	pcs	8	10
SPRI I** trays/load	pcs	8	10

\* = DIN 1/1 = 480 x 250 x 50 mm / 18.90 x 9.84 x 1.97 in  
 \*\* = SPRI I = 450 x 340 x 70 mm / 17.72 x 13.38 x 2.75 in





Getinge is a global provider of innovative solutions for operating rooms, intensive care units, sterilization departments and for life science companies and institutions. Based on our firsthand experience and close partnerships with clinical experts, healthcare professionals and medtech specialists, we are improving the everyday life for people – today and tomorrow.

This document is intended to provide information to an international audience outside of the US.

**Getinge Infection Control AB** · PO Box 69 · SE-305 05 Getinge · Sweden · +46 10 335 00 00 · [info@getinge.com](mailto:info@getinge.com)

[www.getinge.com](http://www.getinge.com)