
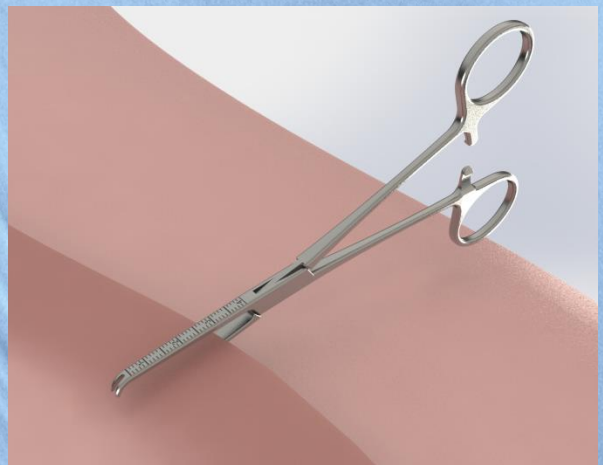


# Marten Facelift Marker

**To mark the incision lines during facelifting procedures**  
Development Partner: Dr. Timothy Marten, USA

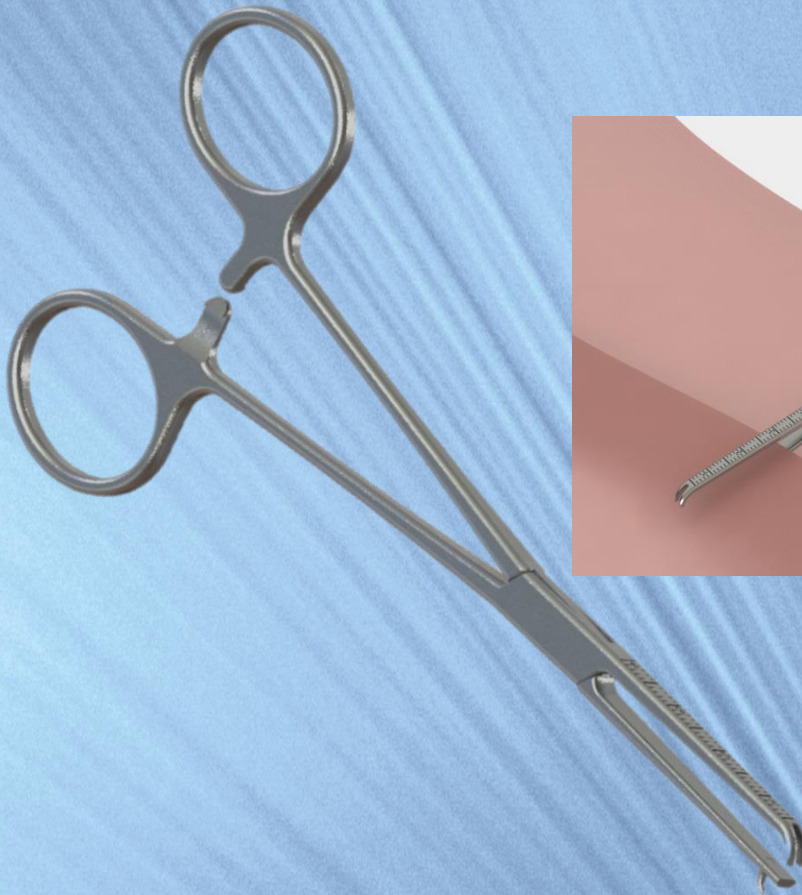

## Product features:

- A **delicate tooth used for skin puncture** ensures a precise definition of the incision line
- The **scale on the upper side of the facelift marker** (in steps of 1mm) ensures to mark constant amounts of skin.
- The **ratchet with only one tooth** applies the appropriate pressure on the skin
- The result is a **clearly visible incision line of redundant skin** during facelifts



|| SURGICAL


**medicon**<sup>®</sup>

Medicon eG · Gänsäcker 15 · D-78532 Tuttlingen  
Tel.: +49 (0) 74 62 / 20 09-0 · Fax: +49 (0) 74 62 / 20 09-50  
E-Mail: sales@medicon.de · Internet: www.medicon.de · Germany

**Marten Facelift Marker**


Application

After dissection and lifting of the facial tissue, the redundant skin is tightened towards the incision edges. The Marten Facelift Marker facilitates to mark the incision lines on the redundant skin.

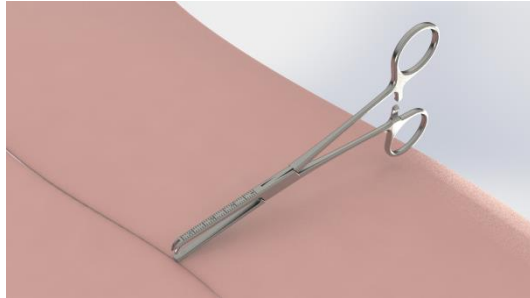


**Step 1**

The delicate tooth on the lower side of the Marten Facelift Marker punctures the skin while the tip of the instrument is placed exactly on the incision edge.

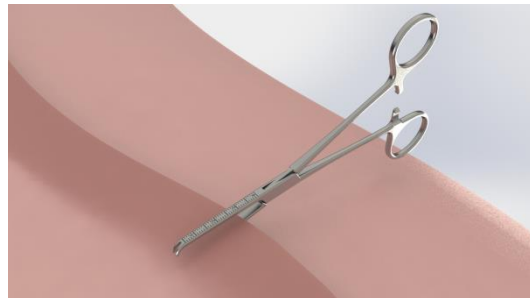

**Step 2**

The redundant skin is pulled into the mouth of the Marten Facelift Marker. The scale on the upper side of the instrument (in steps of 1mm) ensures to mark constant amounts of skin.

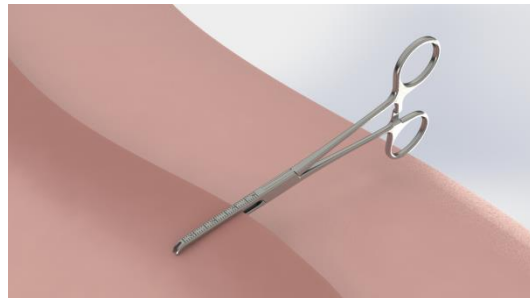


**Step 3**

When closing the instrument the two teeth of the upper mouth part leave an impression on the surface of the skin. The procedure is now repeated until the redundant tissue is marked perfectly along the incision line.



**67.93.31**  
**Marten Facelift Marker 15cm**

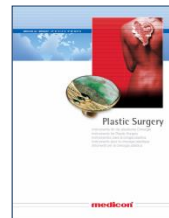

For further interesting products for plastic surgery, please see the MEDICON catalogue  
**Plastic Surgery**  
**450.00.60**  
or visit the MEDION website  
[www.medicon.de](http://www.medicon.de)