

GETINGE ASSURED WASH MONITOR R HOLDER

Product Specification

DESCRIPTION

The Getinge Assured Wash Monitor R Holder is an auxiliary device for routine monitoring of automated wash processes for hollow instruments in washer-disinfectors and ultrasonic washers. Wash Monitor R Holder should only be used with Wash Monitor F/R indicators (in washer-disinfectors) or Wash Monitor U/L indicators (in ultrasonic washers). Wash Monitor F/R or Wash Monitor U/L is inserted into the Wash Monitor R Holder, simulating contaminated rigid lumen instruments.

PERFORMANCE CHARACTERISTICS

The Wash Monitor F/R or Wash Monitor U/L is inserted into the Wash Monitor Holder R. The Holder is then attached to a luer lock connector located on the washer rack. A standard load of hollow lumen instruments is processed as per manufacturer's recommendations. After the wash cycle is complete, results of the cleaning efficacy are displayed on the Wash Monitor F/R and Wash Monitor U/L. The removal of all test soil indicates effective cleaning.

PHYSICAL CHARACTERISTICS

Wash Monitor R Holder made of polished stainless steel.

Dimensions: 197 mm X 11 mm

Weight: 55 g

Substrate Material: Electro polished stainless steel AISI 304

SHELF LIFE

Not Applicable



ORDER INFORMATION

Getinge Assured	Order Number
Wash Monitor Holder R (rigid)	503865000