

## GETINGE ULTRASONICS

Getinge Ultrasonic is a range of high-quality, easy-to-use worktop ultrasonic cleaners that include practically every technical feature available. The models in the new series increase the ultrasonic effect further to achieve great results. The range is available as standard models from 6 to 45 liters.

The functional design of our new units fully meets the cleaning demands of even the toughest continuous operation. Should you require other sizes than the ones mentioned below, please contact your Getinge representative. More information about our products is found in our product specifications and on [getinge.com](http://getinge.com).

The sum of all possibilities:

- Electronic time and temperature control
- Autostart temperature-controlled / temperature pre-selection
- Intelligent & high-performance transducer systems
- Dry run proof ceramic heating
- Degas function for highly efficient acceleration of the degassing process in the cleaning liquid
- Improved cleaning performance with Sweep function
- Safety switch-off
- Lid and an optional basket available for all models.



# THE SPECIAL FUNCTIONS, FEATURES AND ADVANTAGES

The Getinge Ultrasonic range is available in 6 different sizes, ranging from 6 liters up to 45 liters - 6, 10, 15 (non-heated version) 18, 30 & 45 liters. They are all equipped with efficient 37 kHz ultrasonic high-performance transducers of the latest generation.

Example of models:



### Getinge Ultrasonic 60 & 100 (6 & 10 liters)

Ideal machine for low quantities of instruments. Mostly used for de-centralized cleaning in OR och medical practices.



### Getinge Ultrasonic 150 (15 liters)

Specifically designed and suitable for the cleaning of DIN-medicine baskets. Non-heated model.



### Getinge Ultrasonic 300 (28 liters)

Ideal CSSD machine for the cleaning of up to 3 medical norm tray baskets.

#### Intelligent transducers (eco-feature)

The generator always checks and optimizes how to get the power into the system in the best way. In case of heavy load the power is increased and in case of an almost empty machine the power is reduced.

**Optional dry run proof heating** ensures the heating elements are not burned out or over-heated if, for any reason the water level should be too low in the bath.

**Automatic switch off** after 12 hours (for safety reasons)

**User-friendly operating panel** arranged so that no liquid can enter the electronics.

**Newly developed cover** reduces the noise and accelerates the heating-up process.

**Optional Sweep function** guarantees an optimum sound field distribution in the cleaning bath due to a modulation of the frequencies, which makes the cleaning effect equally effective in every part of the bath.

#### LED display for time.

The set cleaning temperature is indicated by a steady light and the actual time is indicated by a blinking light.

#### Turning knob for the pre-selection of the cleaning period

can be turned to permanent operation or short-time operation.

**Autostart** is activated by pushing the start button and allows the user to prepare the bath in advance. The program will not start until the right temperature has been reached.

**Degas function** removes the gases in the water for optimized cleaning efficiency.

**Start/Stop key  
Ultrasound**

**On/Off key**

**Turning knob temperature**

**Cleaning tanks made of cavitation-resistant stainless steel.**

**Marking for filling line.**

**Carrying handles** made of ergonomically shaped plastic.\*

Turning knob on the side allows an **easy draining of the cleaning liquid** through the drain duct on the back of the unit.\*

**LED display temperature** the set temperature is indicated by a steady light and the actual temperature is indicated by a blinking light.



Getinge also offers the Getinge Assured Ultrasonic Monitors, to determine the cleaning effectiveness of the ultrasonic bath.



Getinge also offers the Getinge Clean detergent range suitable for ultrasonic baths.

\*See product specification for more information

**Getinge Infection Control AB**

P O Box 69

SE-305 05 Getinge

SWEDEN

Phone: +46 10 335 00 00

E-mail: info@getinge.com