



INSTRUCTIONS FOR USE for



STERICOOOL HYDROGEN PEROXIDE STERILANT AGENT

ST450 & ST240 & ST030

“For use in STERICOOOL Hydrogen Peroxide Plasma Sterilizer Series only”.

PERSONAL PROTECTIVE EQUIPMENT

WARNING! HANDLING H2O2 STERILANT CARTRIDGES

Sterilizer uses H2O2 as Sterilant consumable. Wear chemical resistant goggles and latex, PVC (vinyl), or nitrile gloves whenever inserting/removing/handling H2O2 Sterilant cartridges.

To signify that the front of the body must be protected by an Apron

To signify that eye protection must be used

To signify that a face mask must be worn

To in case of contact rinse with water

To consult instructions of use



PLEASE SEE MSDS FOR FULL PERSONAL SAFETY AND FIRST AID INFORMATION!

Transportation of Stericool Hydrogen Peroxide Sterilant Agent Cartridges should be made with original packaging provided and all packaging should be transported and handled in upright position. Store in well ventilated place. Keep container tightly closed. Store locked up. Store away from direct sunlight and away from combustibles.

INSTRUCTIONS FOR USE

1. Open the lid of the outer metal box packaging.
2. Wear protective gloves provided in the box.



3. Take out the Sterilant cartridges from the box.
4. Remove Plastic Bag around the bottle.
5. Make sure no vermiculate particles are on the bottle / around the cap.
6. **Remove white cap** of the cartridge before placing the cartridge into the sterilizer.



Do not remove the protective foil on top of the cartridge.

7. Follow the instructions in the sterilizer user manual and place the cartridge inside the sterilizer.

Type	Shelf Life (Closed)	Shelf Life (Opened)
ST030	45 days	7 days
ST240	12 months	30 days
ST450	12 months	49 days