



Sonic Irrigator™ PC+

The standard in surgical instrument reprocessing, this system features advanced monitoring technology to help ensure the “validated” and “clinically-proven” instrument cleaning process is maintained.



DETACHATIP® III
HANDHELD INSTRUMENTS




CORE
ENTREE® II



UNIVERSAL PLUS®
SUCTION / IRRIGATION



To learn more about these and other innovative products, call 800-448-6506 or visit ConMed.com.

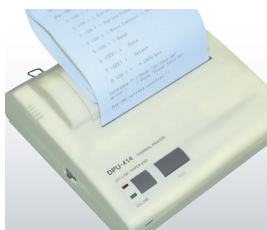

PRINTER

Reprocessing Beyond Reproach

When it comes to effective ultrasonic cleaning, the Sonic Irrigator™ PC+ will clean 12 lumened instruments like DetachaTip® III in a single 15 minute cycle. Featuring Cannulated Pulsed Flush Enhancement (CPE) and monitoring technology, this system automatically adjusts the critical cleaning process where necessary to ensure the validated or clinically-proven process is maintained in accordance with current guidance and international standards. This system features a range of benefits that help OR staff members reprocess instruments effectively and simply.

Simplicity

With the single touch of a button Sonic Irrigator™ PC+ goes to work, thoroughly cleaning instruments inside and out.

Safety

To protect both healthcare staff and patients, the interlocking lid system interrupts the cleaning process ensuring process continuity.

Traceability

In compliance with the requirements outlined in ISO 15883 – a small printer may be connected to the machine enabling the real-time, critical operation parameters and performance indicators to be permanently documented for record keeping and traceability.



DETACHATIP® III VALIDATED

This system has been proven* to effectively clean DetachaTip III scissors, dissectors, graspers and handles. The innovative DetachaTip design features a luer lock cap and port that allows for high lumen flushing without the need to take apart the shafts.

*Data on File

Accessories

This system offers a range of accessories and consumables that help ensure the smooth system operation and maximum performance & operating life:

2nd Basket Option

Use two E-Zee load baskets to improve your instrument processing time and get more from your Sonic Irrigator™ cannulated instrument cleaner.

Hose & Luers

These hoses & luers slips allow for smooth, secure connection to the flush port on the instrument housing, ensuring delivery of the cleaning solution to the device lumen.

Filters

These disposable filters trap large bio-burden particles before they enter the system to prevent costly repairs and keep equipment working at optimal performance levels.

3E-Zyme®

Medisafe's multi-Enzyme detergent is specifically formulated to use with washer disinfectors and optimized for use with ultrasonic equipment.

For accessories and technical support, please call
Medisafe America Customer Service, 877-444-6334.

CANNULATED
INSTRUMENT
CLEANER




HOSE & LUERS



3E-ZYME SOLUTION



525 French Road, Utica, NY 13502

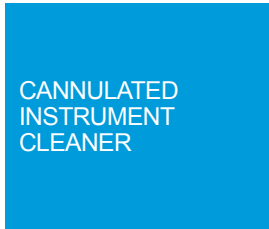

Product Specifications

Use	For internal and external cleaning of fine surgical instruments and other lumen instruments.	Water Supply	Manual fill with tap water, system does not automatically fill (fill / drain hoses provided).
Dimensions	Front to Back: 445mm (17.5")	Environmental	For indoor use at altitudes up to 2000m Operating temperature 10°C to 40°C. Humidity –80% RH max for temperatures up to 31°C, decreasing linearly to 50% RH at 40°C. Designed for Pollution Degree 2, i.e. not totally sealed.
External	Width: 715mm + drain 40mm (28" + 1.5") Height: 345mm (13.5")	Construction	Outer Casing: High grade corrosion resistant stainless steel with brushed finish. Control fascia: Wipe clean polycarbonate Sound Attenuating Lid: Stainless steel
Unit weight	30kg (66lbs) empty		
Capacity	26 liters approx (6.5 US galls approx) 12 cannulated instruments (max) 2 kg of instruments		
Electrical	110~120v, 50-60Hz 5 Watts (standby) 1200 Watts (in use) 1800 Watts (max)		

Ordering Information		
Description	Packaging	Catalog Number
Sonic Irrigator PC+ Series 110v version	1ea	MED1132.1

Please note: The Sonic Irrigator comes complete with a 2.4 meter Cold Water Hose, 2.4 meter Drain Hose, 2.0 meter Power Cord (connected), Data Lead and a User Manual.

Sonic Irrigator is a trademark of Medisafe UK. 3E-Zyme is a trademark of Medisafe UK.



Customer Service: 1-800-448-6506
International Sales: 1-315-797-8375
Fax: 1-800-438-3051

ConMed.com
info@mail.conmed.com

