

GETINGE ASSURED WASH MONITOR F HOLDER

Product Specification

DESCRIPTION

The Getinge Assured Wash Monitor F Holder is an auxiliary device for routine monitoring of automated wash processes for hollow instruments in washer-disinfectors and ultrasonic washers. Wash Monitor F Holder should only be used with Wash Monitor F/R indicators (in washer-disinfectors) or Wash Monitor U/L indicators (in ultrasonic washers). The Wash Monitor F/R or Wash Monitor U/L is inserted into the Wash Monitor F Holder, simulating contaminated flexible hollow lumen instruments.

PERFORMANCE CHARACTERISTICS

The Wash Monitor F/R or Wash Monitor U/L is inserted into the Wash Monitor Holder F. The Holder is then attached to a luer lock connector located on the washer rack. A standard load of hollow lumen instruments is processed as per manufacturer's recommendations. After wash cycle is complete, results of the cleaning efficacy are displayed on the Wash Monitor F/R and Wash Monitor U/L. The removal of all test soil indicates effective cleaning.

PHYSICAL CHARACTERISTICS

Wash Monitor F Holder made of polished stainless steel and silicone.

Dimensions: 1067 mm X 11 mm

Weight: 73 g

Substrate Material: Electro polished stainless steel AISI 304 and silicone conforming quality standards according to European specification ST-EC-60-001

SHELF LIFE

Not applicable



ORDER INFORMATION

Getinge Assured	Order Number
Wash Monitor Holder F (flexible)	503865100