

# Getinge Assured

## Protein Test Flexible Endoscope 2,5 M

The Getinge Assured Protein Test Flexible Endoscope 2,5 M is a quick and easy test used to detect residual protein in hollow lumen surgical instruments.



### Performance Characteristics

The protein test can detect protein residuals to 1 µg sensitivity within 10 seconds. When protein is present, the solution will turn from light brown to blue. This test provides a semiquantitative measure of surface cleanliness; the higher the level of contamination of protein (bioburden), the darker and faster the color changes to blue.

### Physical Characteristics

The Getinge Assured Protein Test Flexible Endoscope 2,5 M contains 10 vials of protein detection solution and 10 swabs.

### Environmental Limitations

Keep away from light and heat exposure.

### Shelf Life

24 months when refrigerated and 6 months when not refrigerated.

### Standards Classification

Consistent with ISO 15883, routine monitoring of washer disinfectors.

### Ordering Information

Getinge Assured	Order Number
Protein Test Flexible Endoscope 2,5M x 2.8mm	503964400
Protein Test Flexible Endoscope 2,5M x 3.7mm	6005500296
Protein Test Flexible Endoscope 2,5M x 1.9mm	6005500297