


Getinge Assured

Helix Test 3.5

The Getinge Assured Helix Test 3.5 verifies air removal, steam penetration, exposure levels and deep vacuum achievement necessary for the sterilization of hollow lumen instruments.

The Helix Test 3.5 is suitable for daily sterilizer monitoring according to EN 867-5. The test is designed to be used in each sterilization cycle as an independent control device.

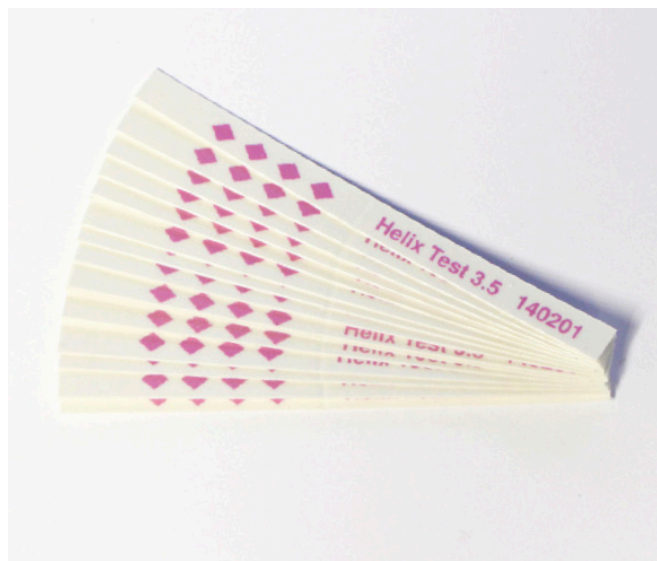

Performance Characteristics

The Getinge Assured Helix Test 3.5 is a reusable device accompanied with single use chemical indicator strips. The indicator strips will change color from pink to black when exposed to appropriate steam sterilization conditions. The adhesive indicator strips can be used for record keeping.

Physical Characteristics

- Lamination: Top
- Dimensions strip: L 77mm x W 6mm
- Packaging: 1 Helix device and 250 indicators

Environmental Limitations

Storage: Store at normal room temperature 10°–38°C and 10-60% Relative Humidity. Protect from moisture and excess humidity. Keep exposed indicators away from light sources.

Shelf Life

24 months. The expiry date is printed on the product packaging.

Ordering Information

Getinge Assured

Helix Test 3.5

Order Number

504052700