

Biological indicator record keeping

Features

Allows users to keep an accurate and detailed log of all biological indicator results

Biological is the complete manual traceability and quality assurance solution that enables CSSD to protect your patients and your facility by making it easy to document all steps in the sterile supply workflow.

This instruction manual describes in detail how traceability throughout the sterile supply workflow,

Once production is complete for the day, each Batch Cycle Record should be attached and filed with a completed Daily Audit Record.

- ✚ Fill in sterilizer number and tick the status boxes.
- ✚ Describe any corrective actions, if needed.
- ✚ Fill in management information.
- ✚ Sign the Daily Audit Record.
- ✚ Fill in autoclave information, name and position of responsible auditor and date.
- ✚ Fill in information based on the daily audits

COLOR CHANGE

- ✚ Biological Media: Purple (pass) and Yellow (fail)
- ✚ Biological Vial Process Indicator: Blue to Black
- ✚ Record Card: Purple to Green or greener than the color standard

INTERPRETATION IMAGES



1 Unexposed

2 Pass

3 Fail (Any presence of purple indicates a fail for the indicator card)

Biological Record Sheet				
Test Date	Sterilizer	Load No.	Initials	COMMENTS
Incubation Date IN:	OUT:	Test Results - +	Control Results - +	
Test Date	Sterilizer	Load No.	Initials	COMMENTS
Incubation Date IN:	OUT:	Test Results - +	Control Results - +	
Test Date	Sterilizer	Load No.	Initials	COMMENTS
Incubation Date IN:	OUT:	Test Results - +	Control Results - +	
Test Date	Sterilizer	Load No.	Initials	COMMENTS
Incubation Date IN:	OUT:	Test Results - +	Control Results - +	
Test Date	Sterilizer	Load No.	Initials	COMMENTS
Incubation Date IN:	OUT:	Test Results - +	Control Results - +	
Biological Results: - No color change in media (sterile) + Media changes to yellow (non-sterile)				